

EasyStand[®] Glider[®]

PNG50187 Medium and PNG50024 Large

Assembly Manual



Please ensure the Assembly Manual stays with the unit at all times.

Table of Contents

Assembly 1


Options & Parts Detail 8

IMPORTANT
EasyStand Warranty Registration
FREE GIFT Upon Completion
(warranty & offer valid in the United States only)

IMPORTANT! Please complete and return within 30 days. To receive the free gift, please fill out all information. Thank You.

To register online go to easystand.com/warranty

**or Scan the
QR Code**

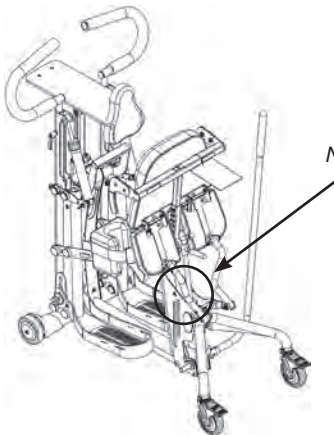

Note: The serial number is located under the seat on the seat column.

Date Purchased: _____

Serial #: _____

Contact Altimate Medical for any information required prior to purchase. This owner’s manual along with other information is freely available by request or by downloading from easystand.com.

SYMBOL/LABEL LEGEND



CAUTION/WARNING/ATTENTION



TEMPERATURE



TYPE BF APPLIED PART



DATE OF MANUFACTURE




PINCHPOINT



SEE INSTRUCTIONS FOR USE



HUMIDITY

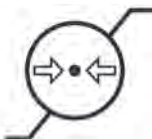

ATMOSPHERIC PRESSURE

SN

SERIAL NUMBER



MANUFACTURER

CE

CONFORMITÉ EUROPÉENNE

Rx only

PRESCRIPTION ONLY

MD

MEDICAL DEVICE




POTENTIALLY HOT SURFACE

Accessible surfaces which may exceed 105°F/41°C and could be a hazard to persons with skin damage or not having sensation.



Waste of electrical and electronic equipment must not be disposed as unsorted municipal waste. It must be collected separately, and must be disposed as per local regulations. Contact your authorized representative for information concerning the decommissioning of your equipment.

CERTIFICATION LABEL/UNIQUE DEVICE IDENTIFICATION LABEL

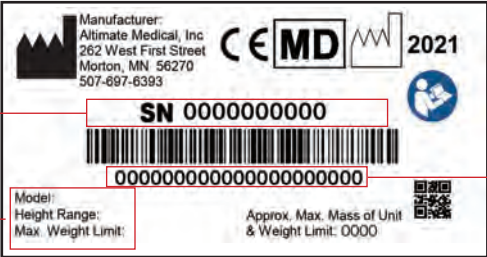

Serial Number

Unique Identifier

Item Number
Item Description

EasyStand® Glider® Assembly

Videos on EasyStand assembly & fitting can be viewed at easystand.com/videos


USE FIG. 1 - FIG. 3 FOR INTERNATIONAL ORDERS ONLY.

FIG. 1- Attach the front caster frame to the front frame with bolts and tighten securely. Laying your unit on the floor side ways may help with lining up the holes.

FIG. 2- Remove nut and flat washer from the bolt on the caster. A flat washer will remain on the bolt on the caster. Place caster, bolt, and flat washer into rear caster frame as shown. Place second flat washer that was removed onto the bolt. Place nut onto bolt and tighten securely with wrench and hex wrench provided in the tool kit. Repeat steps for other caster. Ensure the casters are flush with the rear base frame.


FIG. 3- Place the front frame and the seat assembly on its side, slide the front frame into the opening of the seat assembly as shown. Line up holes and insert all four bolts first, then place nuts onto bolts and tighten securely with hex wrench and 19mm open-ended wrench. Place unit upright on its wheels.

FIG. 4- Remove the ratchet handles and bolts from the glide handles, slide the glide handles into the slots. Tighten securely. If any additional options were ordered, please refer to the drawings on pp. 2-9. Install the seat onto the seat assembly after all other options are added.

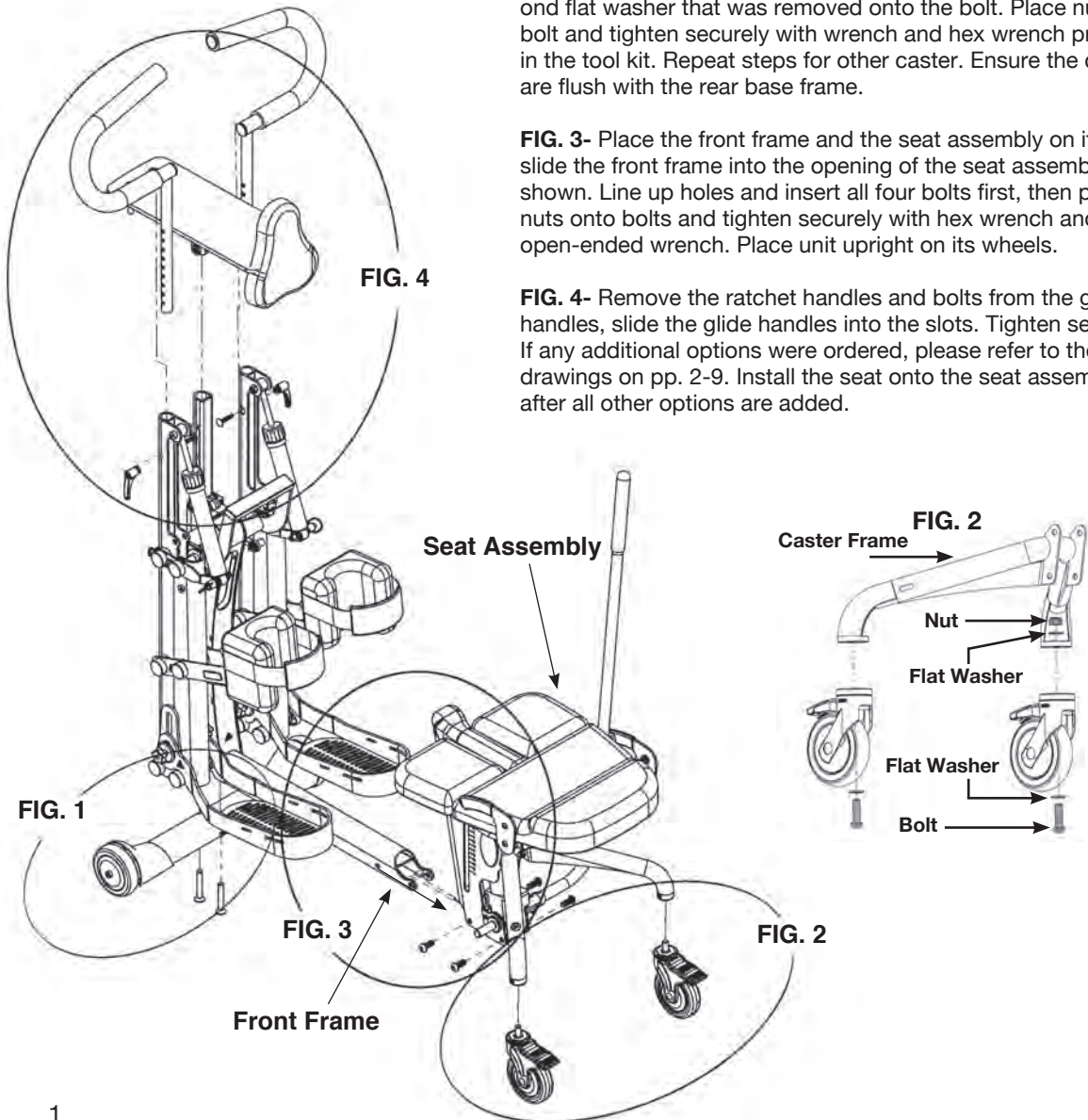

Option Assembly

Back Option assembly for International orders only.

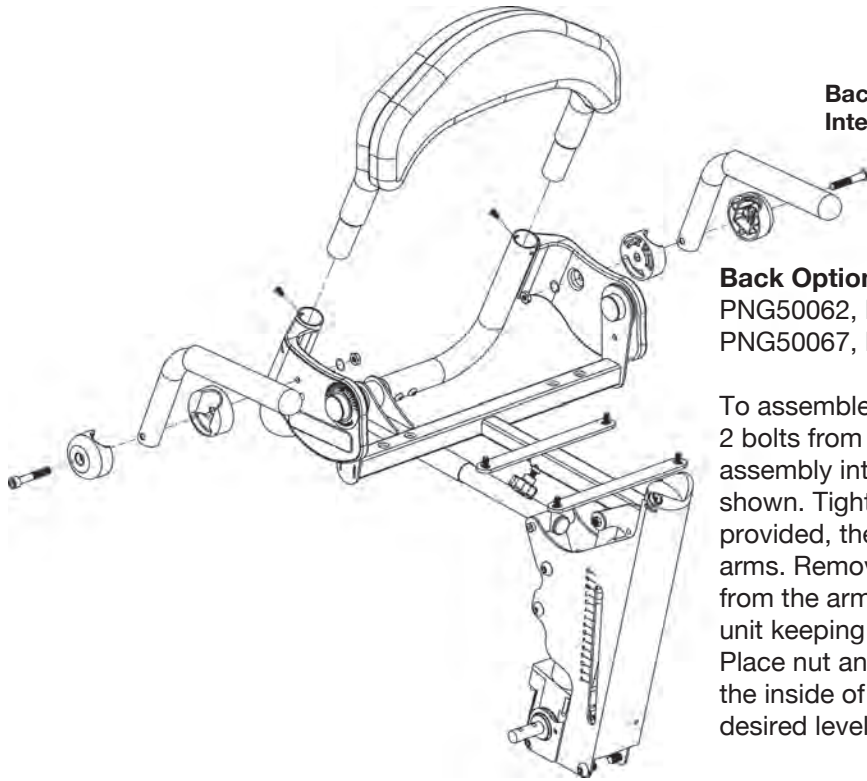

Back Option


PNG50062, PNG50066,
PNG50067, PNG50068

To assemble the back option, remove the 2 bolts from the back, slide the back assembly into the seat assembly as shown. Tighten securely with bolts provided, then add the swing-away arms. Remove the nut and lock washer from the arm. Place the arm onto the unit keeping the arm assembly together. Place nut and lock washer onto bolt on the inside of the unit. Tighten securely to desired level of tension.

Removable Back Option

PNG50171, PNG50172,
PNG50173, PNG50174

To attach the Removable Back to the unit, slide the back assembly into the seat assembly as shown. Insert hardware into back tubes with tabs facing upward and toward the back. Tighten securely with knobs provided.

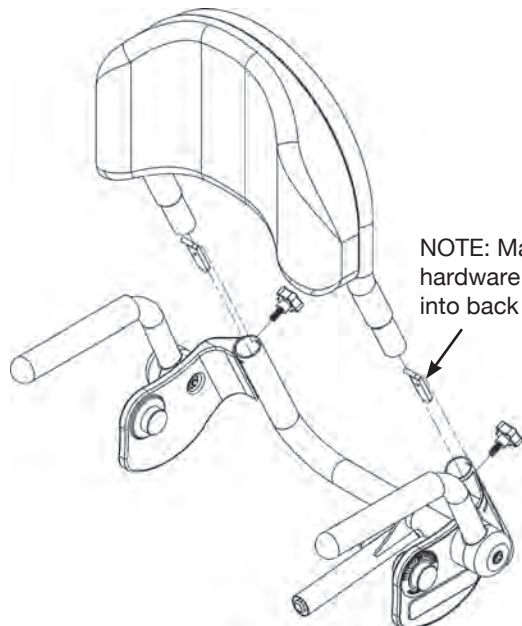

NOTE: Make sure hardware is placed into back as shown.


Glide Handles

Left Side-PNG10194
Right Side-PNG10196
(Large only)



Glide Handles

PNG10177
(Medium only)

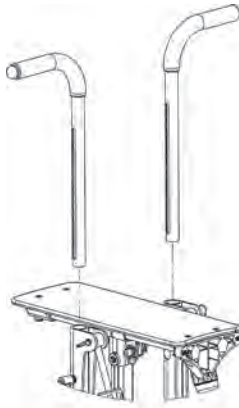

Glide Handle Extensions


PNG50251
(Medium only)



Head Support-Short

21"-30"H
PNG50240


FIG A

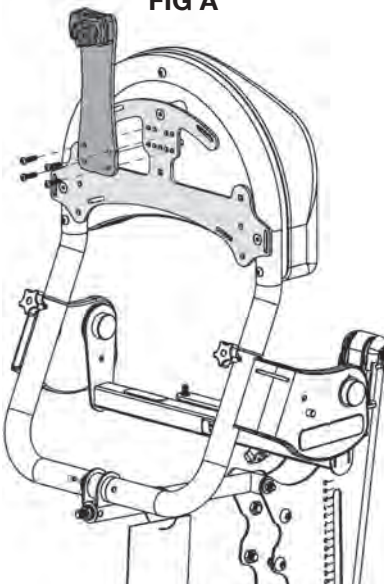

Head Support-Tall

25"-34"H
PNG50037


FIG B

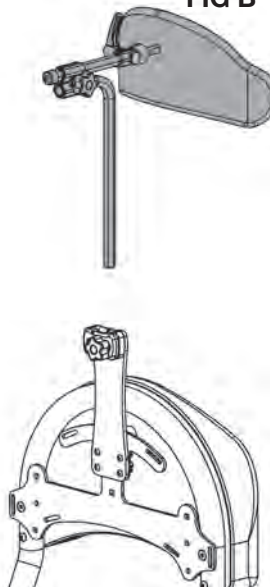

If you ordered the high mount chest vest bracket see p. 6.

FIG A- Place head support bracket onto accessories mounting bracket as shown. Insert 4 bolts and nuts and tighten securely.

FIG B- Insert the head support onto the unit and tighten securely.

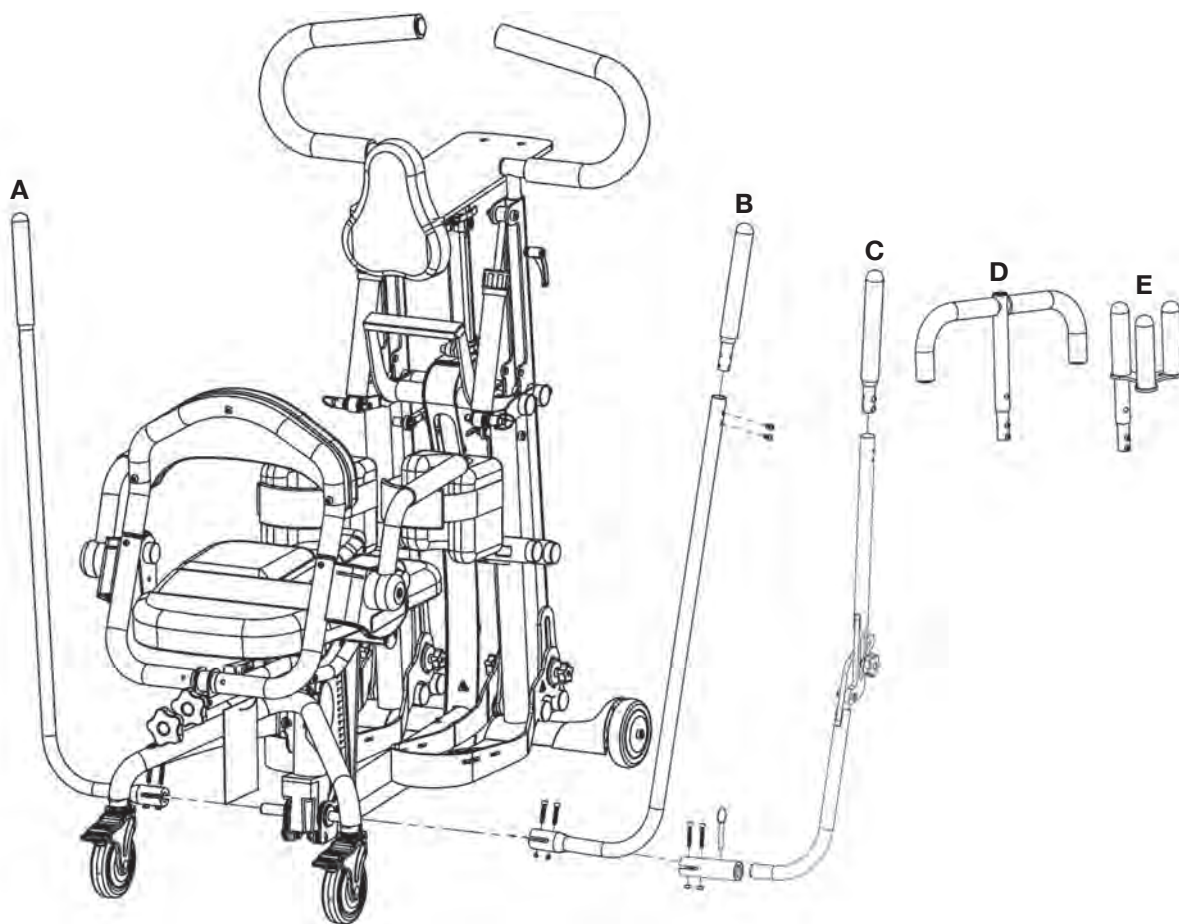

Remove the nuts and bolts from the pump handle. Insert the handle onto the pump. Line up holes and insert bolts, place nuts onto bolts. Tighten securely.

A
Standard
Actuator
Left Handle
 PNG50126


B
Standard
Actuator
Right Handle
 PNG50059

C
Adjustable/
Removable Handle
Universal Left/Right
 PNG50328


D
T-Style
Handle
Extension
 PNG50070

E
Quad Grip
Handle
Extension
 PNG50044


Foot Straps
 PNG30030-15"L
 PNG50505-21"L

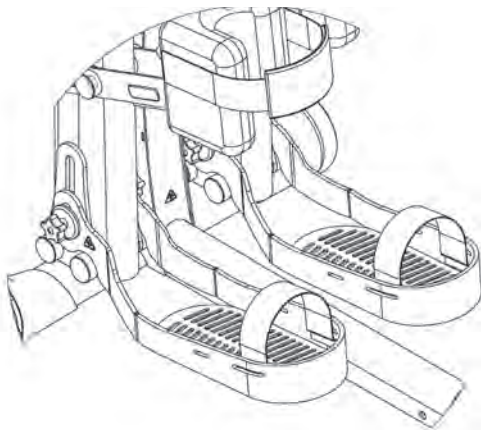


Secure Foot Straps
 PNG30031-15"L
 PNG50506-21"L

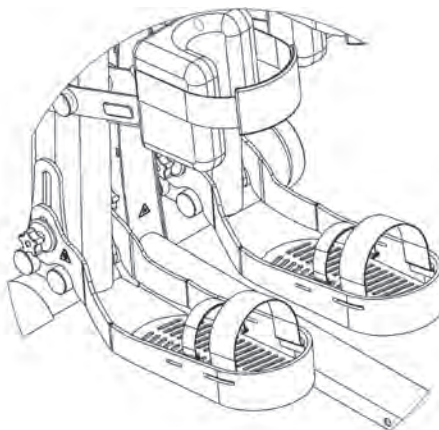

**Airline Buckle
 Hip Belt**
 PNG30028
 26"-55"

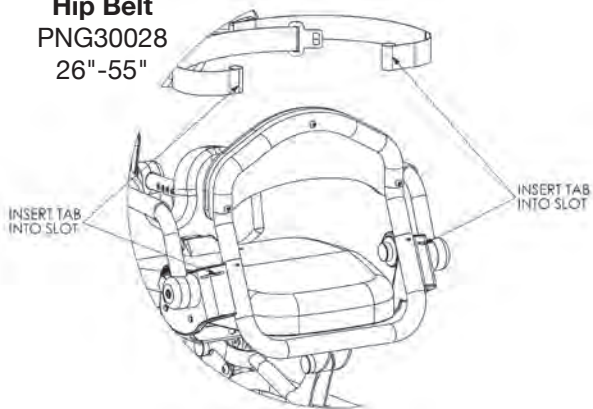

Velcro Hip Belt
 PNG30029-26"-55"
 PNG30306-38"-67"

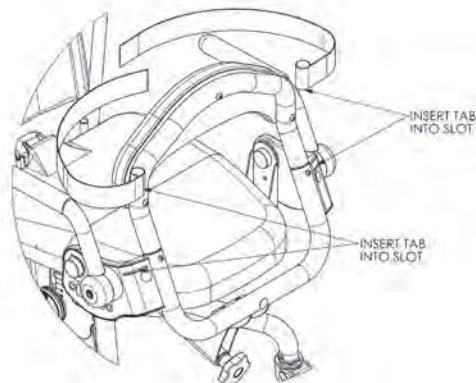

Y-Style Chest Vest

PNG30171-17.5"Lx11.5"W
PNG30021-20.5"Lx12.5"W

Non-Stretch Y-Style Chest Vest




PNG30320-17.5"Lx11.5"W
PNG30324-20.5"Lx12.5"W

X-Style Chest Vest

PNG30170-11"Lx9.5"W
PNG30056-14.5"Lx10.5"W

Non-Stretch X-Style Chest Vest



PNG30318-11"Lx9.5"W
PNG30322-14.5"Lx10.5"W

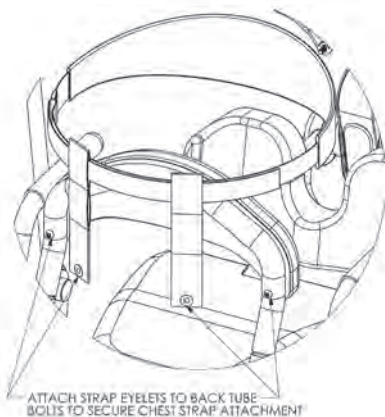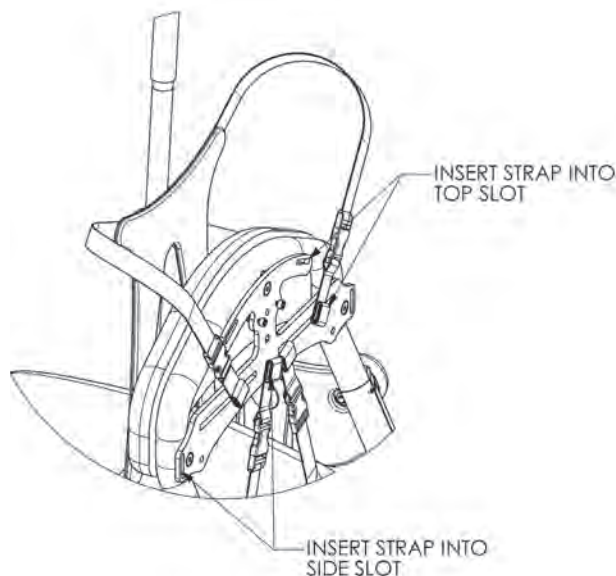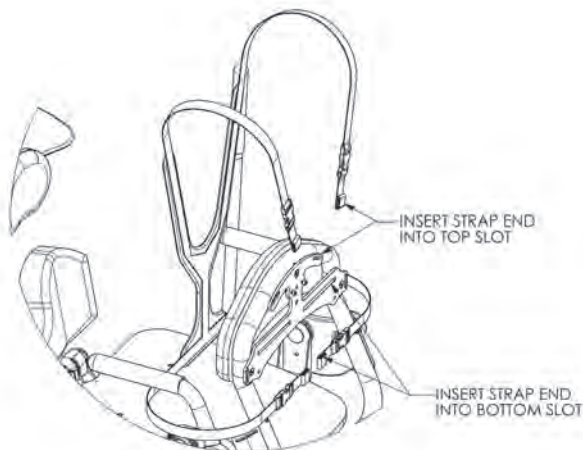

Chest Strap


PNG50192-30"-46"
PNG50193-34"-54"
PNG50504-46"-66"

High Mount Chest Vest Bracket with Head Support Bracket

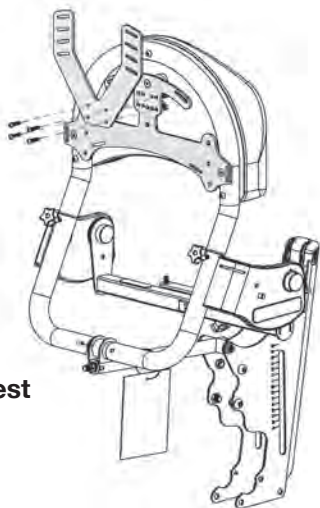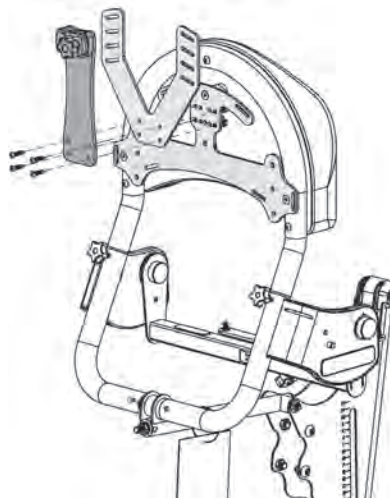

High Mount Chest Vest Bracket PNG11212

If you ordered the high mount chest vest bracket and head support, place the head support bracket and the high mount chest vest bracket on the unit as shown. Insert 4 bolts and nuts and tighten securely. See p. 3 for head support assembly.

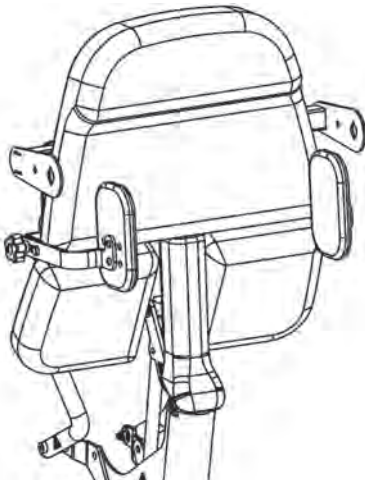

Hip Supports

Glider Medium-PNG50345-9"-15" Range

PNG50374- 11"-17" Range


Glider Large-PNG50345-10"-15" Range

PNG50374- 12"-17" Range

Slide hip supports onto notch and knob under seat.
Tighten knob securely.

If adding hip supports off-unit, remove hole covers from top and bottom of the seat frame

Roho® Seat Glider Medium-PNG50351 Glider Large-PNG50353

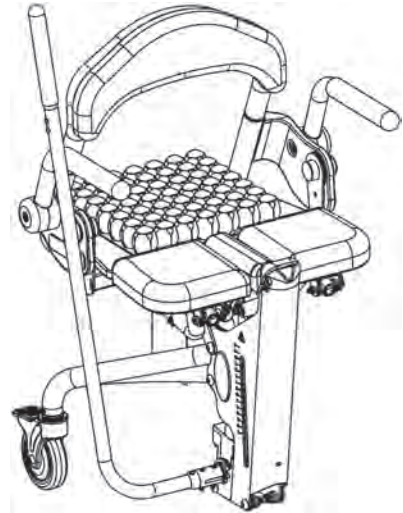

Lateral Supports

with Flat Pads

8"-16.5"W

PNG50535

Lateral Supports

with Flat Pads

11"-19.5"W

PNG50536

Lateral Supports

with Curved Pads

8"-16.5"W

PNG50224


Lateral Supports

with Curved Pads


11"-19.5"W

PNG50176

Slide each lateral support onto notch and knob on the accessories mounting bracket. Tighten knobs securely.

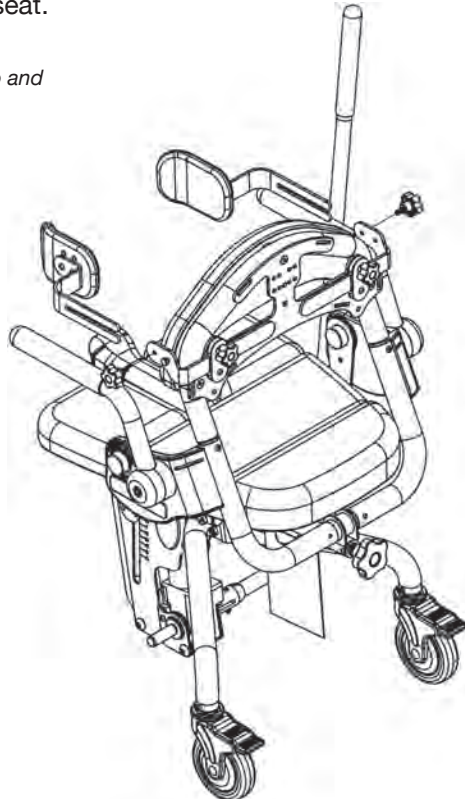

Glider® Options & Parts Detail

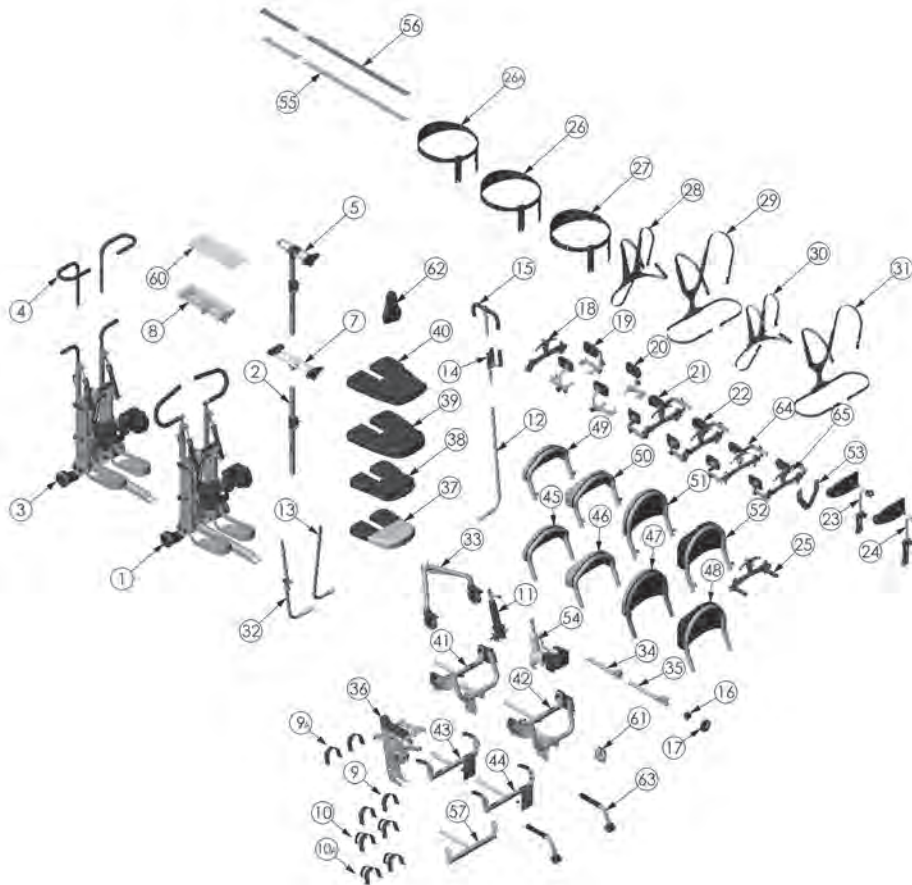

ITEM NO.	PART NO.	NAME	ITEM NO.	PART NO.	NAME
1	NG50024	NEW GENERATION GLIDER ASSEMBLY	34	NG50436	ASMBLY FOLLOWING ARM YOUTH
2	NG50309	TABLE POST ASSEMBLY	35	NG50437	ASMBLY FOLLOWING ARM ADULT
3	NG50187	NEW GENERATION YOUTH GLIDER	36	NG50378	ASMBLY SEAT LIFT GLIDER
4	NG50251	HANDLE ADJ U-SHAPE YOUTH	37	NG50350	GLIDER SEAT ASMBLY YOUTH
5	NG50295	ASMBLY NO-TABLE OPTION	38	NG50351	GLIDER SEAT ASMBLY YOUTH ROHO
6	NG50331	ASSEMBLY ADJUSTABLE PAD BRACKET	39	NG50352	GLIDER SEAT ASMBLY ADULT
7	NG50341	TABLE SLIDE ASSEMBLY	40	NG50353	GLIDER SEAT ASMBLY ADULT ROHO
8	NG50370	TABLE EVOLV GLIDER	41	NG50441	ASMBLY BACK REST YOUTH
9	ng30030	STRAPS FOOT OPTION	42	NG50442	ASMBLY BACK REST ADULT
9A	ng50505	STRAPS FOOT OPTION	43	NG50444	ASMBLY EVOLV NO BACK YOUTH
10	NG30031	STRAPS SECURE FOOT OPTION	44	NG50445	ASMBLY EVOLV NO BACK ADULT
10A	NG50506	STRAPS SECURE FOOT OPTION	45	NG50066	BACK LOW FLAT
11	NG50009	PUMP HYDRAULIC ASSEMBLY LARGE	46	NG50062	BACK LOW W/CONTOUR (V908)
12	NG50059	HANDLE PUMP STANDARD RT	47	NG50067	BACK HIGH FLAT (V909)
13	NG50126	HANDLE PUMP STANDARD LH	48	NG50068	BACK HIGH W/CONTOUR (V910)
14	NG50044	QUAD GRIP OPTION	49	NG50171	BACK REMOVABLE LOW FLAT
15	NG50070	ACTUATOR HANDLE EXTENSION EVOLV LG	50	NG50172	BACK REMOVABLE LOW CONTOURED
16	NG20122	KNOB FEMALE M12 X 1.75	51	NG50173	BACK REMOVABLE HIGH FLAT
17	600002	WHEEL 100mm W/FOLD AWAY HANDLE	52	NG50174	BACK REMOVABLE HIGH CONTOURED
18	NG60034	BRACKET ACCESSORY LG	53	NG11212	HIGH MOUNT CHEST VEST BRACKET
19	NG50161	HIP GUIDE ASSEMBLY MEDIUM	54	NG50010	POWER UP LIFT EVOLV
20	NG50198	HIP GUIDE ASSEMBLY LARGE	55	NG30029	STRAP SEAT BELT NG ADULT
21	NG50224	LATERAL SUPPORTS 8"-16.5"W	56	NG30028	STRAP SEAT BELT W/BUCKLE NG ADULT
22	NG50176	LATERAL SUPPORTS 11"-19.5"W	57	NG50499	ASSEMBLY BEARING SEAT DEPTH
23	NG50240	HEAD REST ASSEMBLY MED	60	NG20019	TABLE GLIDER
24	NG50037	HEAD REST ASSEMBLY LG (V915)	61	80246	SEAT ANGLE LOCATOR
25	NG50040	HANDLE PUSH ASSEMBLY	62	NG30002	CHEST PAD NG GLIDER
26A	NG50504	STRAPCHEST RESTRAINT	63	NG50450	ARM REST ASSEMBLY EVOLV
26	NG50192	STRAP SMALL CHEST RESTRAINT (V904)	64	NG50535	LATERAL SUPPORTS 8"-16.5"W FLAT
27	NG50193	STRAP LARGE CHEST RESTRAINT (V901)	65	NG50536	LATERAL SUPPORTS 11"-19.5"W FLAT
28	NG30056	LARGE CHEST VEST EVOLV			
29	NG30021	VEST CHEST Y SHAPE ASSEMBLY LARGE			
30	NG30170	X-STYLE CHEST VEST MEDIUM			
31	NG30171	Y-STYLE CHEST VEST MEDIUM			
32	NG50328	HANDLE PUMP REMOVABLE/ADJ			
33	NG50058	SEAT REAR CASTER TUBE			

AltimateMedical

Assembly Manual

262 West 1st St.
Morton, MN 56270 USA
Toll Free: 800.342.8968 or 507.697.6393
Fax Free: 877.342.8968 or 507.697.6900
email: info@easystand.com
easystand.com



Medical Product Service GmbH
Borngasse 20
35619 Braunfels
Deutschland



MPS Medical Product Service UK Ltd
Northgate House
Northgate Street
Devizes
Wiltshire England SN10 1JX



MPS Medical Product Service Switzerland, CO/Accumed Sagl
Morbio Inferiore
Viale Serfontana 10
CH-6834 Switzerland



Standing technology should only be used under the guidance of a physician with recommendations for standing program protocol and any medical precautions. Standing programs should be monitored by the attending therapist. AMI maintains a policy of continual product improvement and reserves the right to change features, specifications, and prices without prior notification. Check with AMI for latest info.
FORM GLIDASSEMBAN 012623 REV A Copyright © 2023 Altimate Medical, Inc. All rights reserved. Printed in the U.S.A.