

Kera sit2sit –Assembly Guide



Instructions for assembly of the Kera sit2sit hoist.

Video version available at htsystems.co.nz/learn



1 Unpacking List

- Rolling base with Knee Pad Covers
- Caregiver Handle (80 or 120 model, see handle sticker)
- Chestpad with Arm Cradle Cover and rubber grip
- Backstrap + Owner's Manual
- Laminated Quick Guide (behind knees)



2 Fit Caregiver handle

Position the Caregiver Handle in the Hub and fit the two M8 bolts with washer through Hub and Handle holes, **from the side which the cable runs down**.

Add washer, spring washer then dome nut and tighten as tight as possible with a 5mm hex key and 13mm ring spanner.




3 Fit cable end to Lock Strut

 Careful to ensure the Caregiver Handle does not move when you squeeze the Lock Strut lever.

Note: Cable Path should be down left-hand side and free to move.

Fit the Cable Stop to the right-hand side of the Lock Strut Lever first.

Use thumb to push the Lock Strut lever until the end of the Cable Tube is fitted into the Lock Strut bracket.

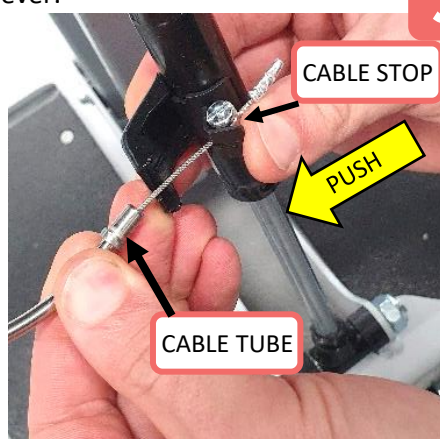

4 Insert Chestpad and grip

Slide Chestpad through Hub, careful at first not to catch the end of plastic strips inside.

Pull up the Index Pin to allow Chestpad to slide into position.

Push the rubber grip onto the Chestpad Handle.



5 Pre-use Check

Perform the Pre-use Check found in the Owner's Manual. Ensure correct function of Caregiver Handle and Lock Lever such that handle locks in position until Lock Lever at top of handle is pulled and handle is free to move.

Ensure Backstrap is with Kera sit2sit, and Quick Instruction card is slipped behind Knee Pad Cover straps.